



HOW TO ASSEMBLE & USE YOUR FACE SHIELD



Watch instructions on video

Video instructions for assembly and use of face shield


<https://www.lacerta.com/lacerta-face-shield>
or follow the directions below

Instructions for the Use of Face Shield and Medical Mask (not N95 mask) During Personal Care Activities

1. Apply medical mask at the beginning of shift and wear for the entire shift.
2. Use face shield and medical mask when providing personal care for a person served (such as bathing, assisting in eating, changing).

To Assemble and Apply Face Shield Over the Medical mask

1. To assemble the face shield you need one face shield and one adjustable strap.
2. Insert the two ends of the adjustable strap into the adjustable strap fit slots. This will help keep the face shield away from the face.
3. Wrap the long plastic strap around your head to measure which slot is the right fit for you. Insert the other end into the slot, you can cut off or bend the excess strap if needed.
4. Wash your hands with soap and water. Use hand sanitizer if soap and water are not available.

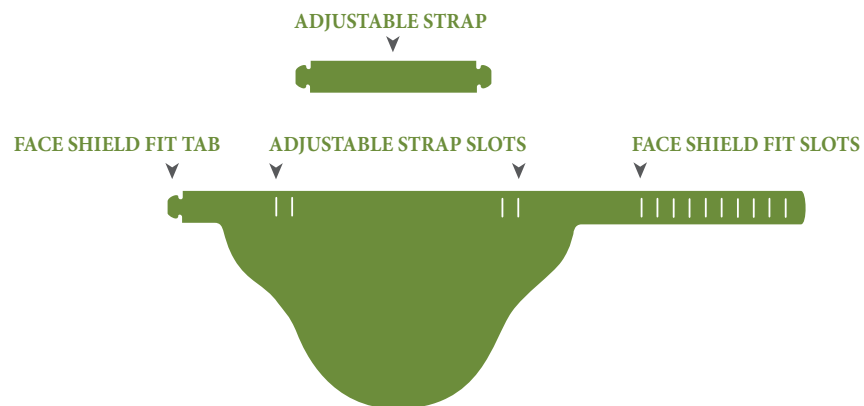

To Remove Face Shield:

1. Remove gloves and wash hands with soap and water. Use hand sanitizer if soap and water are not available.
2. Use clean pair of gloves
3. Grasp face shield by headband and remove from face. Handle only the headband at the sides or back of the face shield. The front is contaminated!
4. Place the face shield in a container where you will clean it.

To clean, disinfect and store Face Shield for future use:

1. Put on gloves.
2. Wipe the inside of the face shield with a cleaning wipe or a clean cloth with mild detergent.
3. Wipe the outside with EPA-approved disinfectant, non-bleach wipe or a clean cloth with mild detergent.
4. If shield is streaked or has a cloudy film, clean mask with soft, damp (with water) cloth or use an alcohol pad.
5. Fully dry (air dry or use clean absorbent towels).
6. Remove gloves and wash hands.
7. Place clean face shield in your personal storage bag, label it, and store in the designated area.

**If face shield appears damaged or torn – discard and obtain a replacement*

Resources



CDC Eye Protection Information

www.cdc.gov/coronavirus/2019-ncov/hcp/ppe-strategy/eye-protection.html

Important FDA Compliance Statement:

- Face Shield for medical purposes made from PET material.
- Intended for single use (As a contingent effort Summit DD recommends that the face shields are disinfected and reused).
- This product has not been FDA cleared or approved.
- This product has been authorized by FDA under an EUA for use by healthcare providers as personal protective equipment.
- This products is only authorized for the duration of the declaration that circumstances justifying the authorization of emergency use under Section 564(b)(1) of the Act, 21 USC 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner.